CAVEX

INSTRUCTIONS

CAVEX
Bite&White

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Instructions for use

DESCRIPTION

Hydrogen peroxide functions as a bleaching agent in Cavex Bite&White Ready 2 Use. It is partly added as a carbamide peroxide to ensure sufficient product stability. Carbamide peroxide is quickly dissolved in oxygen radicals and water. During the bleaching process, the free radicals attack the double carbon compounds, which are the source of tooth discolouration most of the time.

PACKAGING

- Cavex Bite&White Ready 2 Use, 6 pre-filled bleaching trays in a vacuum packaging.

COMPOSITION

Active ingredients:

- 6 % Hydrogen peroxide (partly as carbamide peroxide)
- 0.2% Potassium nitrate

INDICATIONS FOR USE

Cavex Bite&White Ready 2 Use is a gel for whitening the teeth, whereby the colour of natural teeth is bleached. Cavex Bite&White Ready 2 Use may only be supplied by prescription from a dentist. To avoid any mouth or health problems, a consultation with a qualified dentist is recommended before any bleaching treatment.

CONTRA-INDICATIONS

It is recommended that a doctor be consulted in the following cases:

- pregnant and breast-feeding women;
- persons who are being treated for a serious ailment;
- children under the age of 18.

CLINICAL PROCEDURE

- Register the starting colour, evaluate the present health of the teeth and make sure that the expectations of the patient about the final result are realistic.
- Provisional or temporary restorations of the teeth may discolour during the bleaching process; compound or polycarbonate material, however, is not affected. Therefore, patients who have undergone composite

restorations need to be informed that existing composites may have to be replaced, since it is only natural teeth that become whiter.

- Wait two weeks after the most recent bleaching treatment of patients who are in need of an aesthetic or adhesive restoration, allowing the colour to stabilize. All restorations should be intact or properly sealed using the correct temporary materials.
- The use of a tooth paste for sensitive teeth - 2 weeks prior to, and 2 weeks after treatment - should be recommended.
- You should always notify the patient about the sensitivity of the teeth or irritated gingiva during the treatment. The gingiva that comes into contact with the agent may become red or white but will assume a normal colour after a few days.
- To avoid irritation of soft tissue, you should advise the patient to wipe away superfluous gel at the edge of the gingiva, and to rinse the mouth thoroughly after treatment.
- The patient is not allowed to eat, drink or smoke for at least 30 minutes after each treatment.
- Instruct the patient to stop the treatment and to contact you immediately if there is a problem.
- The final colour is achieved approx. 1 week after completion of the treatment.

WARNINGS

- Product contains hydrogen peroxide.
- Avoid contact with the eyes.
- Keep out of reach of children.
- Do not use for overly sensitive teeth.
- Do not eat, drink or smoke when carrying the trays.
- Harmful when swallowed.

FIRST AID INSTRUCTIONS

- If a large amount was swallowed: drink water immediately, do not induce vomiting and consult a doctor.
- If gel has accessed the eyes: pull the eye lids open, rinse the eye with water for at least

15 minutes without interruption and consult a doctor.

- In case of contact with clothing, skin or hair: remove impacted clothing and rinse skin or hair under a running tap.

POSSIBLE SIDE-EFFECTS

Side-effects that might coincide with the use of Cavex Bite&White Ready 2 Use may be the following, though not exclusively:

Sensitivity

During treatment, some patients may experience an increased sensitivity or coldness of the teeth; others may experience a non-specific sensitivity in the teeth, gingiva, tongue, lips or throat. Generally, the symptoms should disappear a few days after the treatment has ended. The treatment may be continued through shorter and less frequent applications until the desired bleaching effect has been reached. If the inconvenience persists, the treatment should be terminated.

Irritation of the gingiva

Trays that extend up to the gingiva may cause a moderate to serious irritation of soft tissue.

STORAGE ADVICE

- Store in refrigerator (<8°C)
- Do not use the material after expiration of the advised shelf life.

DISPOSAL

- Dispose of the product in keeping with local waste disposal regulations.

Your impression is our concern Produced in the Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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