Cavex Holland BV
 P.O. Box 852
 2003 RW Haarlem
 The Netherlands

T +31 23 530 77 00
 F +31 23 535 64 82
 dental@cavex.nl
 www.cavex.nl

Protocol Cavex VacuFormer


Instructions for use

1 Rotate the heating element round to the back, raise the frame all the way to the top using the 2 black handles and then open the frame.

2 Set the timer at ± 10 minutes and set the switch to HEATING (-). The HEATING lamp will now light up. Allow the appliance to preheat ± 2 minutes. The heating element must be red-hot.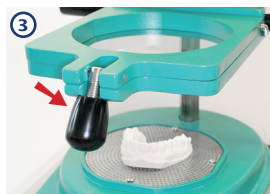

3 Place the laminate in the frame. Close the frame and firmly turn the black knob. Place the plaster mould, preferably trimmed in the shape of a horseshoe, in the centre of the vacuum plate.

4 Rotate the preheated (visibly red-hot) heating element round to the front over the frame.


5 The laminate will now heat up and slowly begin to sag.

6 When the laminate has sagged approx. 3 cm, press the switch to the VACUUM (=) position and push the frame downwards in one swift movement.

7 Rotate the heating element round to the back and allow the vacuum motor to continue running for about 40 seconds. After switching off the vacuum motor wait a further 2 to 5 min. before opening the frame.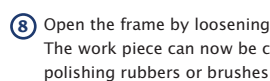

8 Open the frame by loosening the black knob at the front. The work piece can now be cut out and then finished with polishing rubbers or brushes, as desired.

Specific instructions

General

- The frame should only be moved up or down by placing both hands on the black handles
- Visit www.vacuformer.eu and click on Available Information for the FAQs (Frequently Asked Questions) and the appropriate answers

Mouth guard

!! Attention. Do not preheat for this indication !!

- Heat up slowly immediately above the frame with the mouth guard laminate (use an appliance that has cooled down)
- The mouth guard should be cut back to the middle of the 1st molar for more comfort when wearing
- Use a gas burner with a blue flame to give the edges a nice shine

More tips on www.vacuformer.eu

Splint

- Remove the blue foil (2 sides)
- Store in a cool, dry place; the material is slightly hygroscopic
- Before removing the splint from the plaster mould, remove as much excess material as possible in connection with hardening

More tips on www.vacuformer.eu

Bruxism and Splint X-heavy

- Remove the blue foil (2 sides)
- Store in a cool, dry place; the material is slightly hygroscopic
- Cut out with e.g. a small grinding disc

More tips on www.vacuformer.eu

Bleach

- For maximum comfort and to prevent the bleaching gel from prolonged contact with the gingiva, cut out along the edge of the gingiva

More tips on www.vacuformer.eu

Bleach Tray Spacer

- Use Cavex Bleach Tray Spacer to create reservoirs in bleaching trays
 - Only apply Cavex Bleach Tray Spacer to the plaster mould. Only apply to the labial or buccal surfaces of the bleaching elements
 - Apply a film of 1 to 1.5 mm.
 - Keep within approx. 1 mm from the edge of the gingiva
 - Harden with a curing light for 10 sec. per surface

00116AZ

CAVEX YOUR IMPRESSION IS OUR CONCERN

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